

Enabling CGM users with multi-horizon glucose predictive capabilities: The Accu-Chek® SmartGuide Predict App

Christian Ringemann

Accu-Chek® SmartGuide CGM Solution

ACCU-CHEK®



Glucose prediction features



	Glucose Predict	Low Glucose Predict	Night Low Predict
Forecast	Encoder-Decoder NN	Gradient boosting	Gradient boosting
Input	• Time of day • Bolus Insulin • Recent and historic glucose • Carbohydrates	• Recent glucose • Carbohydrates	• Historic glucose excursions • Past nighttime hypoglycaemia • Bolus insulin • Nighttime hypoglycaemia risk in the past
Output	Predicts the future glucose excursion up to 2 h	Low Glucose Risk Prediction within the next 30 min	Nighttime Hypoglycemia Risk Prediction
Visualisation	Glucose curve	Push-Notification	Push-Notification

Validation datasets




	Predict (Clinical Trial)	Replace-BG* (Clinical Trial)	MYSUGR (Real World Data)
No. Subjects	21	226	59
Subject Days	~800	~51,000	~12,000
Diabetes Type	Type 1	Type 1	Type 2
Therapy	CGM+MDI	CGM + Pump	CGM+MDI


Data characteristics of the three employed validation datasets.

* Aleppo G et al. REPLACE-BG: a randomized trial comparing continuous glucose monitoring with and without routine blood glucose monitoring in adults with well-controlled type 1 diabetes. Diabetes Care. 2017;40(4):538-545.

Glucose Predict Results

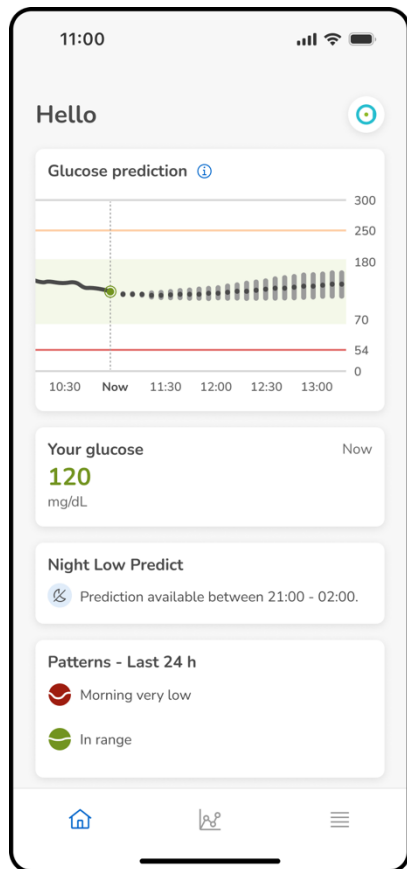

Performance of GP on PREDICT-TEST, REPLACE-BG, and MYSUGR-T2D

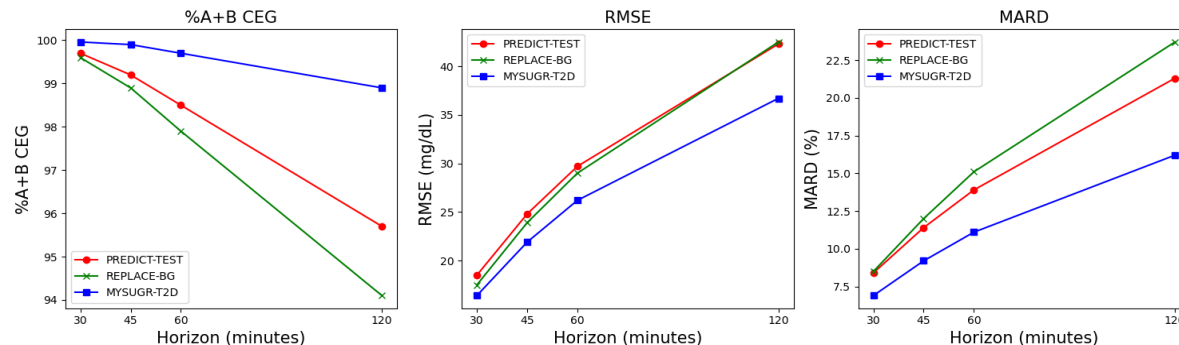


	ForeCasting Horizon [min]	% in Zones A&B of CEG	RMSE** [mg/dL]	MARD*** [%]
Pooled* dataset	30	99.8	17.5	7.9
	45	99.3	23.5	10.9
	60	98.7	28.3	13.3
	120	96.3	40.4	20.3

* Herrero-Vinas P et al. Enhancing the capabilities of continuous glucose monitoring with a predictive app, in press JDST.

** root mean square error

*** mean absolute relative difference

Low Glucose Predict Results

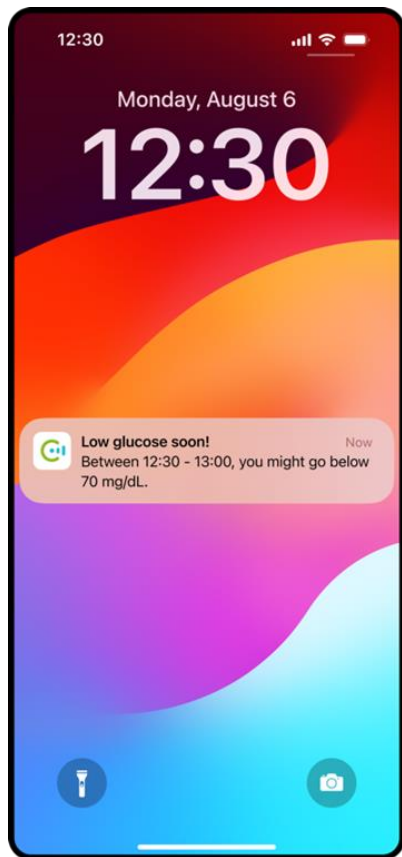


Dataset*	Accuracy [%]	Sensitivity [%]	Specificity [%]	Lead Time [minutes]
PREDICT-TEST	98.6	95.3	98.6	16.2
REPLACE-BG	98.6	95.3	98.7	16.4
MYSUGR-T2D	99.3	95.3	99.3	15.4
POOLED	98.9	95.2	98.9	16.2

- Very similar prediction performance across all three datasets
- On average the low glucose notification is given ~16 minutes before the event occurs

* Herrero-Vinas P et al. Enhancing the capabilities of continuous glucose monitoring with a predictive app, in press JDST.

Night Low Predict Results

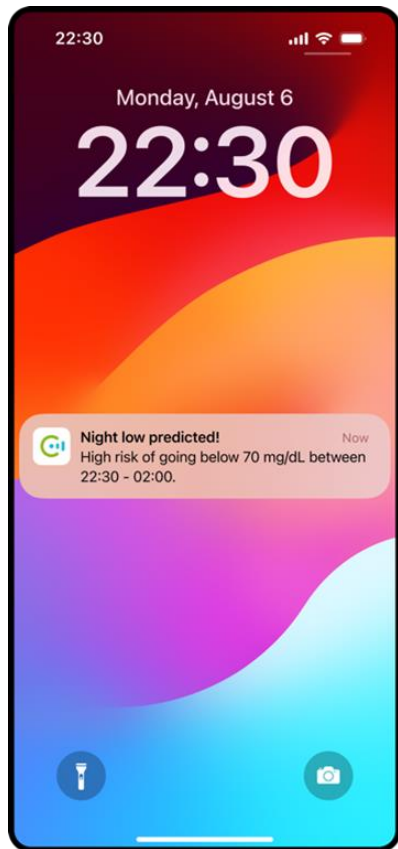

Dataset*	Accuracy [%]	Sensitivity [%]	Specificity [%]	AURoC**
PREDICT-TEST	85.9	67.2	90.3	0.882
REPLACE-BG	77.0	50.7	85.0	0.772
MYSUGR-T2D	93.8	32.1	97.0	0.851
POOLED	86.5	55.3	91.6	0.859

- Slightly lower prediction performance for the Replace BG dataset
- On average for about half of the nights with a potential hypoglycemic event a notification will be pushed to the user

* Herrero-Vinas P et al. Enhancing the capabilities of continuous glucose monitoring with a predictive app, in press JDST.

** area under the receiver operation characteristic

Summary



- The machine learning models powering the three predictive features within the Accu-Chek® SmartGuide Predict app underwent extensive testing using various clinical and real-world datasets, including data from individuals with type 1 and type 2 diabetes on multiple daily injection (MDI) or pump therapy.
- The results from such evaluation provide reassurance that the demonstrated performance should translate into valuable real-world usage of the app, benefiting people with diabetes in their daily management.

Doing now what patients need next

All product names mentioned in this presentation are trademarks of Roche and enjoy legal protection.

© 2024 Roche Diabetes Care
Roche Diabetes Care GmbH
Sandhofer Strasse 116
D-68305 Mannheim, Germany